
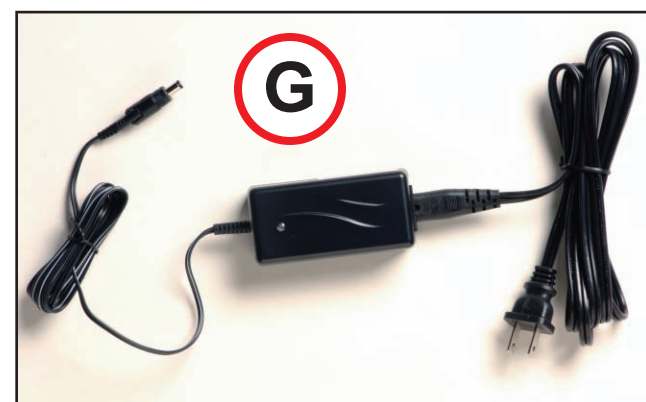


Airpulse PK Diagram

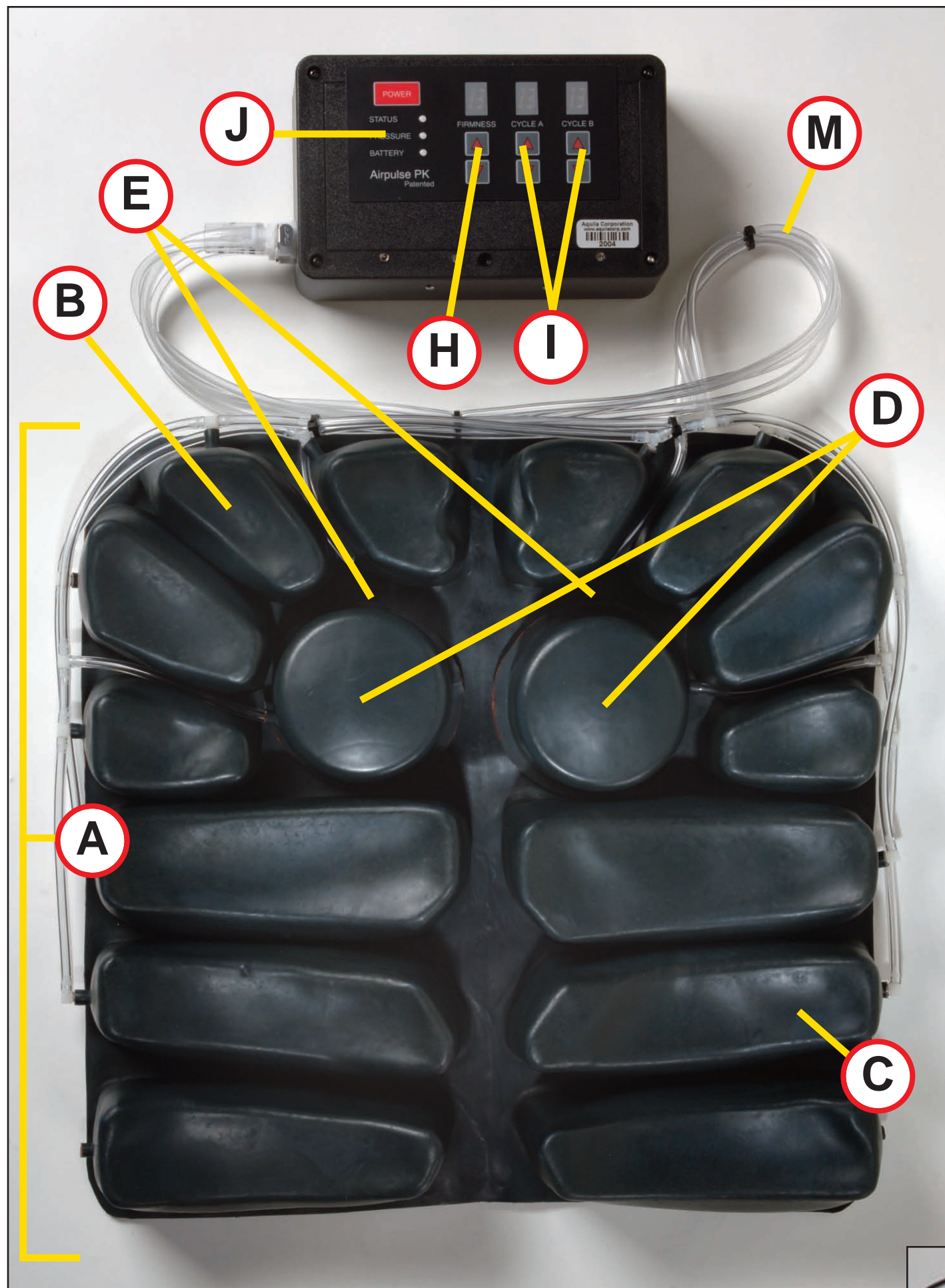

AQUILA CORPORATION

- A** Cushion is made of soft, durable neoprene rubber (no latex) and can be made to any size including for bariatric situations.
- B** Air cells are sculpted to fit the contour of the body to assist positioning and fit.
- C** Air cells have internal bladders for extra strength and are reinforced with foam for safety to prevent a bottoming out event.
- D** Ischial cells have a Velcro backing and can be repositioned as desired or removed to make room for a wound vac.
- E** The round ischial cells can be easily disconnected so they do not inflate for near zero upward pressure under an existing sore in this critical area.
- F** Covers are included for the cushion, tubes, and controller. The cushion cover is stretchable, breathable, and can be machine washed and dried. Incontinent covers are available.
- G** Charger (Double Battery and Hybrid Models) is fully automatic and plugs directly into the controller to charge the battery. No need to remove the battery pack. International models available.
- H** User adjustable firmness setting give you control over cushion inflation pressure.
- I** User adjustable cycle times give you control over frequency of inflation.
- J** Visual and audible indicators for pressure fault and low battery alert.
- K** Power cables (Power chair Model) with a quick disconnect attach directly to the battery terminals and draw power evenly from both chair batteries.
- L** Mini Remote Control (optional) with On/Off switch, system status, low pressure, and battery status indicator lights and beeper.
- M** Air Hose (29 in.) from controller to cushion.
- N** Deluxe Remote Control is an optional accessory that allows you to operate the system without accessing the control box. All system settings can be changed using the Deluxe Remote Control.